# SANITAS

## SFT 75

<b>[GB] [IE] [NI]</b>	<b>Multi functional thermometer</b>	
<b>[MT] [CY]</b>	<i>Instructions for use</i> .....	2
<b>[GR] [CY]</b>	<b>Θερμόμετρο πολλαπλών λειτουργιών</b> <i>Οδηγίες χρήσης</i> .....	12
<b>[IT] [MT]</b>	<b>Termometro multifunzione</b> <i>Istruzioni per l'uso</i> .....	24
<b>Electromagnetic Compatibility</b>		
<b>Information</b> ..... 34		

Service-Hotline:

**[GB]** Tel.: 0800 931 0319

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**[CY]** Tel.: 0800 773 70

**[MT]** Tel.: 800 626 05

**[IE] [NI]** Tel.: 1800 939 235

**[IT]** Tel.: 800 141 010

**CE** 0483

Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

## Table of contents

1. Signs and symbols .....	2
2. Intended use .....	3
3. Notes .....	3
4. Unit description .....	4
5. Display description .....	5
6. Initial use .....	5
7. Basic settings .....	5
8. What you should know about using this thermometer .....	6
9. Measuring .....	6
9.1 Measuring body temperature on the forehead .....	7
9.2 Measuring body temperature in the ear .....	7
9.3 Measuring object temperatures .....	8
9.4 Displaying stored measurements .....	8
10. Replacing the battery .....	9
11. Cleaning and storage .....	9
12. Disposal .....	9
13. What if there are error messages? .....	10
14. Technical data .....	10
15. Warranty/Service centre .....	11

### Included in delivery

Check that the delivery has not been tampered with and make sure that all components are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Services address.

## 1. Signs and symbols

The following symbols appear in these instructions for use and on the device:

	<b>WARNING</b> Warning notice indicating a risk of injury or damage to health.
	<b>IMPORTANT</b> Safety note indicating possible damage to the unit/accessory.
	<b>Note</b> Note on important information.
	Observe the instructions for use

	Application part, type BF
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE
	The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EEC on medical products.
	Manufacturer
<b>IP 22</b>	Protected against solid foreign objects 12.5 mm in diameter and larger, and against drops of water when the housing is angled up to 15°
	Permissible storage temperature and humidity
	Permissible operating temperature and humidity
<b>SN</b>	Serial number

## 2. Intended use

This multi functional thermometer is intended solely for measuring forehead temperature, ear temperature and object temperature. It allows you to quickly and easily measure body temperature and object temperature. Values are saved automatically. The multi functional thermometer is suitable for domestic use.

Use the device for the intended purpose only and in the manner specified in these instructions for use. Any form of improper use can be dangerous. The manufacturer is not liable for damage resulting from improper or careless use.

This device complies with EU Directive 93/42/EEC concerning medical devices, the Medizinproduktegesetz (German Medical Devices Act), the ASTM (American Society for Testing and Materials) E 1965 - 98 and the European Standard EN 12470-5: Clinical thermometers – Part 5: Performance for infrared ear thermometers (with maximum device) and the European Standard EN60601-1-2, and is subject to particular precautions with regard to electromagnetic compatibility. Please note that portable and mobile HF communication systems may interfere with this unit. More details can be requested from the stated Customer Services address or found at the end of the instructions for use.

## 3. Notes

### Safety notes

#### WARNING

- To measure temperature, insert the sensor tip of the thermometer carefully into the ear.
- Use of the thermometer on different persons can be inappropriate in the event of certain acute infectious diseases because of the possible spread of germs despite cleaning and disinfection. If you have any doubts, please consult your doctor.
- Keep the device away from children. Choking hazard! Children may swallow the packaging material or the forehead/cover cap.
- Children must not be allowed to use the device. Medical products are not toys.
- Check before each use that the lens is intact. If it is damaged, contact your retailer or the service address.
- The device has been designed for practical use, but is not a substitute for a visit to the doctor.

## Notes on handling batteries

- If your skin or eyes come into contact with battery fluid, rinse the affected areas with water and seek medical assistance.
- **⚠ Choking hazard!** Small children may swallow and choke on batteries. Store the batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect batteries from excessive heat.
- **⚠ Risk of explosion!** Never throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a relatively long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, split or crush the batteries.


## General notes

- Repairs must only be carried out by authorised service centres, otherwise the guarantee is invalidated.
- This thermometer is a sensitive electronic device. Handle it with care and do not subject it to any mechanical impacts.
- Do not expose the device to direct sunlight.
- The thermometer is NOT waterproof. For this reason, direct contact with water or other liquids must be avoided.
- The SFT 75 multi functional thermometer is only designed for the measuring area stated in the instructions for use.
- After each use, clean the measuring tip with a soft cloth moistened with disinfectant.


- Before making a claim, please check the batteries first and replace them if necessary.
- Repairs must only be carried out by authorised service centres; otherwise, the guarantee is invalidated.
- If you still have any questions, please contact your retailer.

## 4. Unit description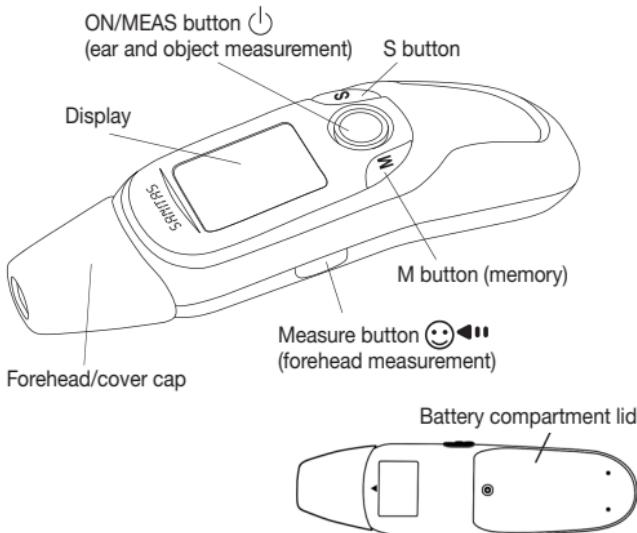


## 5. Display description

- 1 Date display
- 2 Time display
- 3 Measurement in progress
- 4 Measuring unit  
Celsius/Fahrenheit
- 5 Object temperature mode
- 6 Temperature value
- 7 Button tone symbol
- 8 Battery level
- 9 Memory space number display
- 10 Forehead temperature mode
- 11 Ear temperature mode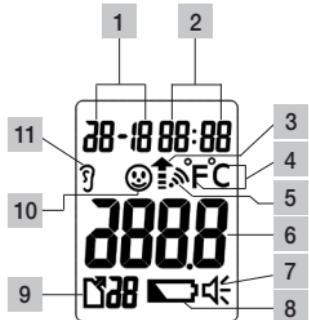

3. The current temperature unit setting flashes on the display. Use the **S** button to select the temperature unit ( $^{\circ}\text{C}$  or  $^{\circ}\text{F}$ ). Use the **M** button to confirm.

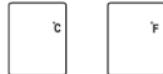4. The year display flashes on the display. Use the **S** button to select the current year. Use the **M** button to confirm.

5. The month display flashes on the display. Use the **S** button to select the current month. Use the **M** button to confirm.

6. The day display flashes on the display. Use the **S** button to select the current day. Use the **M** button to confirm.

7. The hour display flashes in the display. Use the **S** button to select the current hour. Use the **M** button to confirm.

8. The minute display flashes in the display. Use the **S** button to select the current minute. Use the **M** button to confirm.

9. The device stores the basic settings and automatically switches off.

### Activating/deactivating the button tone

You can activate/deactivate a button tone on the device.

When the button tone is activated, a beep sounds every time a button is pressed and after every measurement.

To activate the button tone, ensure that the device is switched on and then briefly press the **S** button. The loudspeaker symbol appears on the display. To deactivate the button tone again, press the **S** button once more. The loudspeaker symbol disappears from the display.

## 6. Initial use

The batteries are already inserted in new devices. Before first use, remove the protruding plastic insulating strip from the battery compartment. The multi functional thermometer will then switch on automatically.

## 7. Basic settings

You can adjust the following basic settings on the multi functional thermometer: Temperature unit (Celsius/Fahrenheit), date and time.

1. Press the **ON/MEAS** button to switch on the device.
2. Hold down the **S** button for five seconds.

## 8. What you should know about using this thermometer

There are different thermometers for taking temperature measurements in different parts of the body:

- Ear/forehead thermometer (this thermometer, for measuring in the ear or on the forehead area)
- Rod thermometer (for rectal [in the back passage], axillary [under the arm] or oral measurement [in the mouth])

### Note

Taking the temperature gives a measurement that provides information about a person's current body temperature.

If you are uncertain about interpreting the results or if the value is abnormal, please consult your doctor. This also applies in the case of slight temperature changes if there are other symptoms of illness such as agitation, severe sweating, flushed skin, fast pulse rate, tendency to collapse etc.

Temperatures measured with different thermometers cannot be compared with one another.

You should therefore tell your doctor (or bear in mind if you are diagnosing yourself) what thermometer you used to take your temperature and in what part of the body.

The temperature of a healthy person is affected by various factors: the person's individual metabolism, their age (body temperature is higher in babies and toddlers and decreases with age). Greater temperature fluctuations occur faster and more often in children, e.g. due to growth spurts), their clothing, the ambient temperature, the time of day (body temperature is lower in the morning and increases throughout the day towards evening), the preceding physical and, to a lesser extent, mental activity.

The temperature varies depending on the part of the body where the measurement is taken. The difference can be between 0.2°C (0.4°F) and 1°C (1.8°F) for a healthy person.

The normal temperature ranges are as follows:

- On the forehead: 35.8°C (96.4°F) to 37.6°C (99.7°F), measured with a forehead thermometer
- In the ear: 36.0°C (96.8°F) to 37.8°C (100°F), measured with an ear thermometer
- In the back passage: 36.3°C (97.3°F) to 37.8°C (100°F), measured with a conventional thermometer
- In the mouth: 36.0°C (96.8°F) to 37.4°C (99.3°F), measured with a conventional thermometer

In order to track temperature change, always take the measurements in the same part of the body.

## 9. Measuring

### Note

Check before each use that the lens is intact. If it is damaged, contact your retailer or the service address.

Bear in mind that the multi functional thermometer needs to have been in the room in which the measurement is taken for at least 30 minutes before use.

Before each measurement, make sure that the device is in the appropriate mode for the measurement that you wish to take.

Mode	Ear	Forehead	Object	Button tone activated
Display				

## 9.1 Measuring body temperature on the forehead


### Note

Please note that the forehead/temples must be free from sweat and cosmetics and that taking vasoconstrictive medication and skin irritations can distort the result when measuring temperature on the forehead.

1. Hold down the **ON/MEAS** button  for one second to switch on the multi functional thermometer.

2. Place the measuring head with the forehead/cover cap fitted on the temple.

3. Keep the side **measure button**  pressed and move the multi functional thermometer smoothly over the forehead to the other temple.

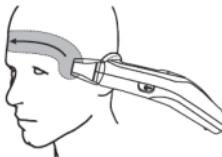4. Then release the **measure button** .

The measured value appears on the display and the end of the measuring is signalled by a short beep (if button tone is activated).

5. If the measured value is within the normal range ( $<38^{\circ}\text{C}/100.4^{\circ}\text{F}$ ), the green LED lights up for 3 seconds. If the measured value is higher ( $\geq 38^{\circ}\text{C}/100.4^{\circ}\text{F}$ , high temperature), the red LED is illuminated.

6. The device switches off automatically after 60 seconds.

7. Clean the device after every use as described in the "Cleaning and storage" chapter.

## 9.2 Measuring body temperature in the ear


### WARNING

- Some people produce different readings in their left and right ear. In order to record temperature changes, always measure a person's temperature in the same ear.
- The built-in ear thermometer may be used by children only under adult supervision. Measurement is usually possible over the age of 6 months. In infants under 6 months, the ear canal is still very narrow so the temperature frequently cannot be recorded and the measurements displayed are often too low.
- The measurement must not be taken in an ear affected by inflammatory diseases (e.g. discharging pus or secretion), after possible ear injuries (e.g. eardrum damage) or in the healing period after operative procedures. In all of these cases, please consult your doctor.
- Use of the multi functional thermometer on different persons can be inappropriate in the event of certain acute infectious diseases because of the possible spread of germs despite cleaning and disinfection. If you have any doubts, please consult your doctor.
- If you have been lying on one ear for some time, the temperature is slightly raised. Wait a little while or measure in the other ear.
- As ear wax can affect the measurement, you should clean the ear before measuring if necessary.

1. Hold down the **ON/MEAS** button  for one second to switch on the multi functional thermometer.

2. Remove the forehead/cover cap by gently turning it to the right (1) and then pulling it off forwards (2).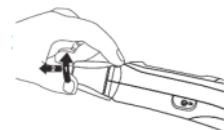

3. Make sure that the sensor tip and also the ear canal are clean. As the ear canal is slightly curved, you have to pull the ear slightly up and backwards before inserting the sensor tip. This is important so that the sensor tip can be pointed directly at the eardrum.

#### Positioning on small children

On infants under 1 year of age:

- Lay down the infant such that one ear is pointing upwards.
- Pull the ear gently backwards before you carefully insert the sensor tip into the ear.

In small children older than 1:

- Stand slightly to the side behind the child.
- Pull the ear gently up and backwards before you carefully insert the sensor tip into the ear.


4. Insert the sensor tip carefully and hold down the **ON/MEAS** button  for one second.

5. The measured value appears on the display and the end of the measuring is signalled by a short beep (if button tone is activated).
6. If the measured value is within the normal range (<38°C/100.4°F), the green LED lights up. If the measured value is higher ( $\geq 38^\circ\text{C}/100.4^\circ\text{F}$ , high temperature), the red LED is illuminated.
7. The device switches off automatically after 60 seconds.
8. Clean the device after every use as described in the "Cleaning and storage" chapter.

## 9.3 Measuring object temperatures

1. Hold down the **ON/MEAS** button  for one second to switch on the multi functional thermometer.
  2. Then hold down the **M** button and the **S** button simultaneously for 3 seconds to switch to object temperature mode.  appears on the LCD display.
  3. Hold down the **ON/MEAS** button  and point the sensor tip towards the object or liquid to be measured (never immerse in liquids), maintaining a distance of 3 cm.
  4. Release the **ON/MEAS** button .
- The measured value appears on the display and the end of the measuring is signalled by a short beep (if button tone is activated).
5. To exit object temperature mode, hold down the **M** button and the **S** button simultaneously again for 3 seconds.
  6. The device switches off automatically after 60 seconds.
  7. Please note that the displayed temperature is the measured and not the adapted surface temperature. It is not the same as the forehead/ear temperature.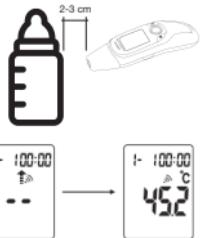

## 9.4 Displaying stored measurements

The device automatically stores the values from the last 30 measurements. When 30 memory spaces are exceeded, the oldest value is deleted. The stored values can be called up as follows:

1. Hold down the **ON/MEAS** button  for one second to switch on the multi functional thermometer.
2. Press the **M** button. The most recent measurement appears on the display.
3. To switch between the individual stored measurements, press the **M** button.
4. The device switches off automatically after 60 seconds.

## 10. Replacing the battery

Before replacing the battery, ensure that the device is switched off. When the batteries get weak, the battery warning symbol appears on the display . It is still possible to measure temperature. However, the batteries must be replaced as soon as possible.

If the battery warning symbol  and  are flashing on the display, the batteries must be replaced immediately. If the batteries are too flat, then the thermometer will switch off automatically.

To change the batteries, follow the instructions below:

1. Use a screwdriver to turn the screw on the battery compartment lid.
2. Remove the battery compartment lid.
3. Take the flat batteries out of the battery compartment.
4. Insert the new batteries into the battery compartment until you hear and feel them slot into place. Ensure when inserting the batteries that the correct polarity (+/-) is observed.
5. Place the battery compartment lid back on.
6. Use a screwdriver to re-tighten the screw on the battery compartment lid.

- To clean the entire device, use a soft cloth slightly moistened with a mild soapy solution.
- Do not use any harsh cleaning products.
- Always store the thermometer with the forehead/cover cap on.
- If prolonged storage is intended, you should remove the batteries.
- The device must not be stored or used at an excessively high or low temperature or humidity (see technical data), in sunlight, in association with an electrical current or in dusty locations. Otherwise inaccuracies can occur.

## 12. Disposal

The empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.

The codes below are printed on batteries containing harmful substances:

Pb = Battery contains lead,  
Cd = Battery contains cadmium,  
Hg = Battery contains mercury.


For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the unit at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

## 11. Cleaning and storage

- Clean the sensor tip and the forehead/cover cap after each use. Use a clean cloth or cotton bud that can be moistened with disinfectant, alcohol or warm water.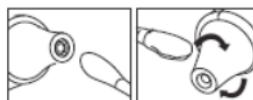

## 13. What if there are error messages?

Error message	Cause	Solution
	The recorded temperature is higher than 1) Ear/forehead thermometer mode: 43°C (109.4°F) 2) Object temperature mode: 100°C (212°F)	Operate the thermometer only between the specified temperature ranges. If necessary, clean the sensor tip. In the event of a repeated error message, contact your retailer or Customer Services.
	The recorded temperature is lower than 1) Ear/forehead thermometer mode: 34 °C (93.2 °F) 2) Object temperature mode: 0°C (32°F).	
	The operating temperature is not in the range from 15°C to 35°C (59°F to 95°F).	Operate the multi functional thermometer only between the specified temperature ranges.

If you cannot find a solution to your problem here, contact our Customer Services.

## 14. Technical data

Name and model	SFT 75
Measurement range	Ear/forehead thermometer mode: 34 °C – 43 °C (93.2 °F – 109.4 °F) Object temperature mode: 0 °C – 100 °C (32 °F – 212 °F)
Laboratory measurement accuracy	Ear thermometer mode: $\pm 0.2^\circ\text{C}$ ( $\pm 0.4^\circ\text{F}$ ) from 35°C to 42°C (95°F to 107.6°F), outside of this measurement range $\pm 0.3^\circ\text{C}$ ( $\pm 0.5^\circ\text{F}$ ) Forehead thermometer mode: $\pm 0.2^\circ\text{C}$ ( $\pm 0.4^\circ\text{F}$ ) from 35°C to 42°C (95°F to 107.6°F), outside of this measurement range $\pm 0.3^\circ\text{C}$ ( $\pm 0.5^\circ\text{F}$ ) Object temperature mode: $\pm 1.5^\circ\text{C}$ ( $\pm 2.7^\circ\text{F}$ ) at <30°C (86°F); $\pm 5\%$ (in 3 cm) at $\geq 30^\circ\text{C}$ (86°F)
Interval between 2 measurements	At least 5 seconds
Clinical repeat precision	Ear: Children, 1 – 5 years: $\pm 0.08^\circ\text{C}$ ( $\pm 0.14^\circ\text{F}$ ) Adults: $\pm 0.07^\circ\text{C}$ ( $\pm 0.13^\circ\text{F}$ ) Forehead: Children, 1 – 5 years: $\pm 0.07^\circ\text{C}$ ( $\pm 0.13^\circ\text{F}$ ) Adults: $\pm 0.08^\circ\text{C}$ ( $\pm 0.14^\circ\text{F}$ )
Measurement units	°Celsius (°C) or °Fahrenheit (°F)
Operating conditions	15°C to 35°C (59°F to 95°F) with a relative humidity of up to 85% (non-condensing)
Storage conditions	-25°C to 55 °C (-13 °F to 131 °F) with a relative humidity of up to 85% (non-condensing)
Dimensions	132 x 37 x 40 mm

Weight	61 g (without batteries)
Battery	2 x 1.5V AAA (LR03)
Memory	For 30 measurements

The serial number is located on the device or in the battery compartment.

If requested, please send the faulty product to the following address:  
NU Service GmbH  
Lessingstraße 10 b  
89231 Neu-Ulm  
Germany

## 15. Warranty/Service centre

We offer a 3-year product warranty from the date of purchase against material and production faults. The warranty does not cover:

- Damage due to improper operation
- Wearing parts
- Cases of personal negligence on the part of the customer
- The event that the device has been opened by an unauthorised workshop.

The warranty does not affect the customer's statutory rights. To assert a warranty claim within the warranty period, the customer must provide proof of purchase. The warranty claim must be submitted within a period of 3 years from the date of purchase to Hans Dinslage GmbH, Uttenweiler, Germany.

**In case of complaints, please contact our service department using the following contact details:**

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# ELECTROMAGNETIC COMPATIBILITY INFORMATION

**Table 1**

## Guidance and manufacturer's declaration-electromagnetic emissions

The SFT75 Multi-function Thermometer is intended for use in the electromagnetic environment specified below. The customer or the user of the SFT75 Multi-function Thermometer should assure that it is used in such an environment.

<b>Emissions test</b>	<b>Compliance</b>	<b>Electromagnetic environment – guidance</b>
RF emissions CISPR 11	Group 1	<p>The SFT75 Multi-function Thermometer uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.</p>
RF emissions CISPR 11	Class B	<p>The SFT75 Multi-function Thermometer is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.</p>
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Not applicable	

**Table 2**

## Guidance and manufacturer's declaration – electromagnetic immunity

The SFT75 Multi-function Thermometer is intended for use in the electromagnetic environment specified below. The customer or the user of the SFT75 Multi-function Thermometer should assure that it is used in such an environment.

<b>IMMUNITY test</b>	<b>IEC 60601 test level</b>	<b>Compliance level</b>	<b>Electromagnetic environment – guidance</b>
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floor should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV air for input/output lines	Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1kV differential mode ±2kV common mode	Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% $U_T$ (>95% dip in $U_T$ for 0.5 cycle)  40% $U_T$ (60% dip in $U_T$ for 5 cycles)  70% $U_T$ (30% dip in $U_T$ for 25 cycles)  5% $U_T$ (>95% dip in $U_T$ for 5 sec)	Not applicable	Mains power quality should be that of a typical commercial or hospital environment. If the use of the SFT75 Multi-function Thermometer requires continued operation during power mains interruptions, it is recommended that the SFT75 Multi-function Thermometer be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

**NOTE:**  $U_T$  is the a.c. mains voltage prior to application of the test level.

**Table 3**

## Guidance and manufacturer's declaration – electromagnetic immunity

The SFT75 Multi-function Thermometer is intended for use in the electromagnetic environment specified below. The customer or the user of the SFT75 Multi-function Thermometer should assure that it is used in such an environment.

<b>IMMUNITY test</b>	<b>IEC 60601 test level</b>	<b>Compliance level</b>	<b>Electromagnetic environment – guidance</b>
Conducted RF IEC 61000-4-6	3Vrms 150 kHz to 80 MHz	3Vrms	<p>Portable and mobile RF communications equipment should be used no closer to any part of the SFT75 Multi-function Thermometer, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p><b>Recommended separation distance:</b></p> $d = 1.2 \sqrt{P}$ $d = 1.2 \sqrt{P} \text{ 80 MHz to 800 MHz}$ $d = 2.3 \sqrt{P} \text{ 800 MHz to 2.5 GHz}$ <p>Where <math>P</math> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and <math>d</math> is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,<sup>a</sup> should be less than the compliance level in each frequency range.<sup>b</sup></p>  <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p>
Radiated RF IEC 61000-4-3	3V/m 80 MHz to 2.5 GHz	3V/m	

**NOTE 1** At 80 MHz and 800 MHz, the higher frequency range applies.

**NOTE 2** These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Field strengths from fixed transmitters, such as base stations from radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast can not be predicted theoretic call with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the SFT75 Multi-function Thermometer is used exceeds the applicable RF compliance level above, the SFT75 Multi-function Thermometer should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the SFT75 Multi-function Thermometer.

Over the frequency range 150kHz to 80MHz, field strengths should be less than [V1] V/m.

**Table 4**

**Recommended separation distances between portable and mobile RF communications equipment and the EQUIPMENT or SYSTEM-for EQUIPMENT and SYSTEMS that are not LIFE-SUPPORTING**

Recommended separation distances between portable and mobile RF communications equipment and the SFT75 Multi-function Thermometer

The SFT75 Multi-function Thermometer is intended for use in an electromagnetic environment in which radiated RF distances are controlled. The customer or the user of the SFT75 Multi-function Thermometer can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the SFT75 Multi-function Thermometer as recommended below according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz $d = 1.2 \sqrt{P}$	80 MHz to 800 MHz $d = 1.2 \sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

**NOTE 1** At 8- MHz and 800 MHz, the separation distance for the higher frequency range applies.

**NOTE 2** These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Hans Dinslage GmbH, Riedlinger Str. 28, 88524 Uttenweiler, Germany

758.861-0417